making work easy

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# Contents

<b>1</b>	<b>Introduction</b>	<b>2</b>
1.1	Symbols used	2
1.2	EU Conformity	2
<b>2</b>	<b>Safety</b>	<b>3</b>
2.1	Proper Use	3
2.2	Ambient conditions for safe operation	3
2.3	Ambient conditions for storage and transport	3
2.4	Hazard and Warning Information	3
2.4.1	General Information	3
2.5	Information about electromagnetic compatibility (EMC)	4
2.6	Authorised Individuals	4
2.7	Liability Exclusion	4
<b>3</b>	<b>Product Description</b>	<b>4</b>
3.1	General Description	4
3.2	Assembly and Controls	5
3.3	Standard Delivery	6
3.4	Accessories	6
<b>4</b>	<b>Commissioning</b>	<b>6</b>
4.1	Setup	6
4.1.1	Benchtop Unit	6
4.1.2	Wall Mounting	6
4.1.3	Setup Recommendations	6
4.2	Connection and Starting	6
4.3	Remove Shipping Restraint	6
<b>5</b>	<b>Operation</b>	<b>7</b>
5.1	Cartridge Installation and Replacement	7
5.1.1	Select mixing program	7
5.2	Extrusion	8
5.3	Fill Level Indicator and Remaining Amount	8
5.3.1	Fill Level Indicator	8
5.3.2	Remaining Amount Detection	8
<b>6</b>	<b>Cleaning / Maintenance</b>	<b>9</b>
6.1	Cleaning the Interior	9
6.2	Pressure Disc Replacement	9
6.3	Unit Cover Replacement	9
6.4	Fuse Replacement	9
6.5	Spare Parts	9
<b>7</b>	<b>Troubleshooting</b>	<b>10</b>
<b>8</b>	<b>Technical Specifications</b>	<b>11</b>
<b>9</b>	<b>Warranty</b>	<b>11</b>
<b>10</b>	<b>Disposal Information</b>	<b>11</b>
10.1	Disposing of Consumables	11
10.2	Instrument Disposal	11
10.2.1	Disposal Information for EU Nations	11

# 1 Introduction

en

## 1.1 Symbols used

The following symbols are employed in these instructions and on the unit itself:

**Follow the Instructions for Use!**

**Caution**  
**Serious risk of injury!**

**Electrical current**  
This indicates a hazard due to electrical current.

**Attention**  
Failure to observe the associated information can result in damage to the unit.

**Note**  
*This provides the operator with useful information to make working with the unit easier.*

The operating instructions can be found in electronic format on our website.  
Visit [www.renfert.com/p915](http://www.renfert.com/p915). Enter the following item number: 25 6200 0000.

**Please observe the operating time.**

**After no more than 2 minutes of operation, you must pause for at least 5 minutes.**

**After immediate processing of 3 complete cartridges (approx. 30 impressions, middle-sized impression tray) a 2 hours quiescent period should be observed. During this time, the unit may remain connected.**

The EU regulations applicable to the product are adhered to.

Within the EU, this device is subject to the provisions of the WEEE directive.

**Manufacturer**

**Date of manufacture**

**Medical device**

**Rx only To be used by specialist staff.**

### ► List, particular attention should be paid

- List

- Subordinate List

⇒ Instructions / appropriate action / input / operational sequence:

You will be asked to carry out the action in a specified order.

- ◆ Result of an action / reaction of the device / reaction of the program:

The unit or program reacts as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

## 1.2 EU Conformity

Renfert GmbH hereby declares that this product:

**Dynamix®speed 2**

Item number: 25 6200 0000, 25 6200 1000, 25 6200 3000

complies with the following European regulations:

Regulation (EU) 2017/745

## 2 Safety

Using these operating instructions as a starting point, instruct all operators of the unit with regard to the area of application, the possible hazards during operation, and the proper operation of the unit.

Please have these operating instructions readily available for the operators.

**In case of complaints and events in connection with the medical device, please inform Renfert immediately. Renfert will investigate the matter and report serious incidents to the authorities in the Member States concerned.**

### 2.1 Proper Use

The Dynamix<sup>®</sup>speed 2 is designed solely for the extrusion of 2-component precision impression materials.

With the Dynamix<sup>®</sup>speed 2 it is possible to use 5:1 cartridges or pouches in support cartridges from various manufacturers.

Dynamic mixing nozzles with hex drive are used to mix the material.

**Always comply with all safety and processing instructions provided by the manufacturer!**

**Only the mixing nozzles specified by the manufacturer may be employed.**

### 2.2 Ambient conditions for safe operation

Safe operation of the unit can only be ensured under the following ambient conditions:

- Indoors;
- Up to an altitude of 2,000 m above sea level;
- At an ambient temperature range between 5 - 40 °C [41 - 104 °F] \*);
- At a maximum relative humidity of 80 % at 31 °C [88 °F], dropping in a linear manner to 50 % relative humidity at 40 °C [104 °F] \*);
- With mains power where the voltage fluctuations do not exceed 10 % of the nominal value;
- Under contamination level 2 conditions;
- Under over-voltage category II conditions.

\*) Between 5 – 40 °C [41 – 104 °F], the unit can be operated at a relative humidity of up to 80 %. At temperatures between 31 – 40 °C [88 – 104 °F], the humidity must decrease proportionally in order to ensure operational readiness (e.g., at 35 °C [95 °F] = 65 % humidity; at 40 °C [104 °F] = 50 % humidity). The unit may not be operated at temperatures above 40 °C [104 °F].

### 2.3 Ambient conditions for storage and transport

Ensure the following ambient conditions during storage and transport:

- Ambient temperature - 20 – + 60 °C [- 4 – + 140 °F],
- Maximum relative humidity 80 %.

### 2.4 Hazard and Warning Information

#### 2.4.1 General Information

- ▶ **Only to be used indoors. The device is only intended for dry operation and should not be used or stored outdoors or in wet conditions.**
- ▶ **The device may only be operated using a mains cable with the country-specific plug system.**
- ▶ **The device may only be plugged into outlets which are connected to the protective conductor system.**
- ▶ **The mains plug must be easily accessible.**
- ▶ **Pull out the mains plug to disconnect the equipment from the mains supply.**
- ▶ **Check connection cables (such as power cords), tubes and housing (i.e. the key pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing. Do not operate equipment with damaged connection cables, hoses or housing parts or other defects!**

- ▶ The equipment may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.
- ▶ Always unplug the device from the wall outlet before beginning any work on the unit's electrical components.
- ▶ Note any hazard or warning information provided by the material manufacturer.
- ▶ Do not operate in an explosion-prone area.
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with.  
For Germany these are MPBetreibV in relation with DIN EN 62353 (VDE0751).
- ▶ Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.
- ▶ Information on REACH and SVHC is available on our website [www.renfert.com](http://www.renfert.com), in the Support area.

## 2.5 Information about electromagnetic compatibility (EMC)

"Information about electromagnetic compatibility (EMC) can be found in a separate section at the end of these instructions."

## 2.6 Authorised Individuals

This product may not be used by minors under the age of 14.

Only properly trained individuals may operate and service the unit.

Any repairs not specifically described in these operating instructions may only be performed by a qualified electrician or authorized dealers.

## 2.7 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is employed for any purposes other than those cited in the operating instructions;
- ▶ The product is altered in any way other than those alterations described in the operating instructions;
- ▶ The product is repaired by other than an authorised facility or if any but Renfert OEM parts are employed;
- ▶ The product continues to be employed, despite obvious safety faults or damage;
- ▶ The product is subjected to mechanical impacts or is dropped.
- ▶ We do not accept any liability for damage caused by mixing tips, foil bags or cartridges splitting.

# 3 Product Description

## 3.1 General Description

The Dynamix<sup>®</sup>speed 2 unit for mixing impression materials facilitates extrusion and mixing of 2-component dental impression materials.

5:1 cartridges or foil bags in cartridge holders from different manufacturers can be used in the unit.

Impression materials marked with the symbols  or  can be extruded in double the speed with the relevant mixing programme in the Dynamix<sup>®</sup>speed 2.

The mixing programs control the feed rate of the pressure discs and the rotational speed of the hexagonal head.

Only mixing nozzels should be used which are designated to the impression material by the manufacturer.

### 3.2 Assembly and Controls

- 1 Fill level indicator
- 2 "Impression tray" advance key (fast extrusion speed)
- 3 "Dispenser" advance key (slow extrusion speed)
- 4 Retract key
- 5 Yellow LED, residual level indicator
- 6 Green LED, unit ON
- 7 Red LED, unit cover open
- 8 Unit cover
- 9 Pressure discs
- 10 Spindle covers
- 11 Spindles
- 12 Catch
- 13 Hexagonal end
- 14 Cartridge bed
- 15 Control keys and LED display for mixing programs
- 16 Mains switch
- 17 Fuse
- 18 Mains connectionr
- 19 Power cord
- 20 Cartridge / Cartridge with tubular bag
- 21 Mixing nozzle
- 22 Service connection

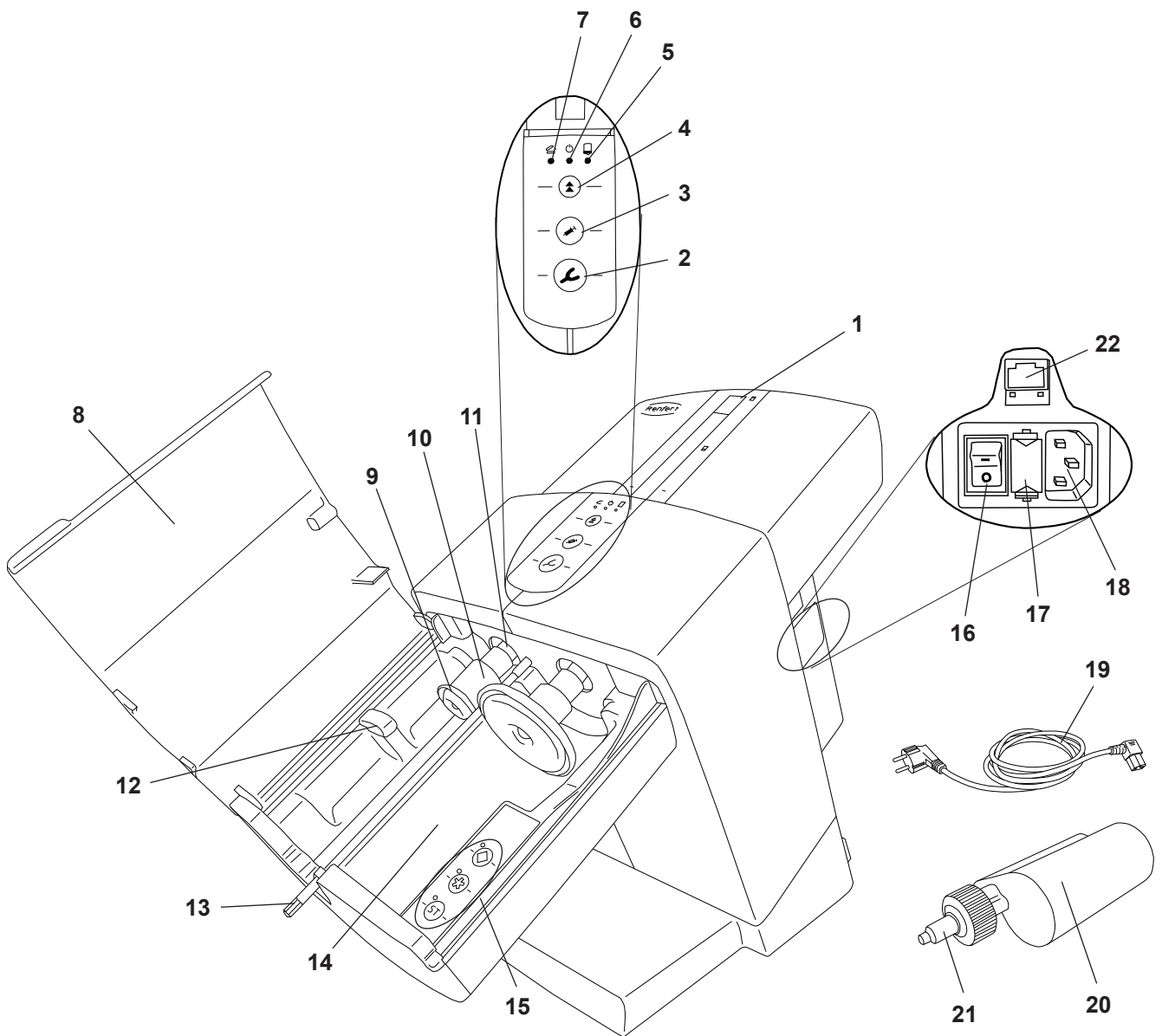
Fig. 1

### 3.3 Standard Delivery

- 1 Dynamix®speed 2 mixing unit
- 1 Power cord
- 1 Operating instructions

### 3.4 Accessories

- 25 6200 0100 Wall mounting kit

## 4 Commissioning

### 4.1 Setup

The equipment can be used as a benchtop unit, although it is also suitable for wall mounting.

#### 4.1.1 Benchtop Unit

Select a stable underlay on which to set the unit up.

#### 4.1.2 Wall Mounting

With the aid of the wall mounting kit (refer to the accessories), the unit can be securely mounted on a wall. The mounting instructions are included in the mounting kit.

#### 4.1.3 Setup Recommendations

Operate the unit at room temperature 18 - 24 °C [64 – 75 °F].

When setting up the unit, please note the following:

- Do not place the unit under a heat source.
- Do not place the unit in front of open windows.
- Do not set the unit up where it will be subjected to direct sunlight.
- Do not expose the unit to high humidity.

### 4.2 Connection and Starting

**The equipment may only be operated if the information on the rating plate conforms to the specifications of your local mains power supply.**

- ⇒ Connect the provided power cord (19, Fig. 1) to the mains connection socket (18).
- ⇒ Plug the power cord into the wall outlet.
- ⇒ Switch the unit on at the mains power switch (16).

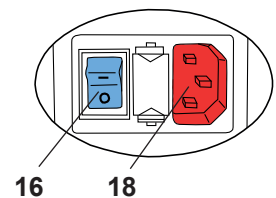
Fig. 2

### 4.3 Remove Shipping Restraint

The shipping restraint must be removed before the unit may be operated:

- ⇒ The unit has been connected and switched on as described in Section 4.2.
- ⇒ Close the unit cover (8, Fig. 1).
- ⇒ Press the retract key (4).
  - ◆ The pressure discs (9, Fig. 1) retract to their home position where they stop.
- ⇒ Open the unit cover.
- ⇒ Remove the shipping restraint (23).

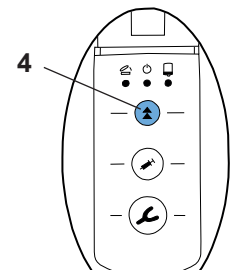
Fig. 3

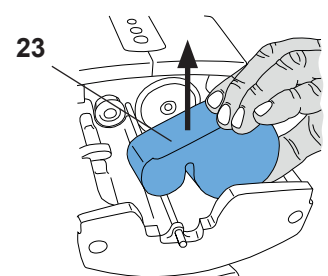
Fig. 4

The unit is now ready for use.

# 5 Operation

## 5.1 Cartridge Installation and Replacement

A cartridge can only be installed or replaced if both pressure discs (9, Fig. 1) are in their home position.

⇒ Close the unit cover (8, Fig. 1).

⇒ Press the retract key (4).

- ◆ The pressure discs return to their home position and automatically stop.

**i** *Just before the end, the retraction speed reduces. Only open the unit cover once the pressure discs have reached their home position and have stopped.*

**i** *If the return path of the pressing table is stopped by pressing the motion button, the table will not re-continue its path automatically. In order to continue it is necessary to press the return path button again.*

⇒ Open the unit cover (8, Fig. 1).

⇒ Place cartridge (20) in unit attached. Please make sure that the cartridge is fitted into place under the catch (12).

⇒ Select mixing program (15), cf. also section 5.1.1.

Information on the mixing program can be found on the cartridge in the form of an „asterisk“ \* or „diamond“ ◆ symbol.

All impression materials which are not marked with these symbols must be processed with the programme <sup>ST</sup>.

⇒ Close the unit cover (8, Fig. 1).

The cover will only close if the cartridge has been properly installed. The red “Unit cover open” LED (7, Fig. 1) will remain on until the unit cover has been properly closed.

⇒ Please refer to the instructions of the casting material manufacturer when preparing and levelling the cartridge systems.

**!** **Use only mixing nozzles recommended by the material manufacturer. The use of other mixing nozzles may cause malfunction.**

### 5.1.1 Select mixing program

The mixing program is selected by pressing the program keys (15). The selection of a particular mixing program is confirmed by the green LED lamp next to the relevant program key.

The mixing programs marked with an asterisk \* or diamond ◆ may only be used with special casting materials and cartridge/mixing nozzle systems which carry these symbols.

Any casting materials and cartridge/mixing nozzle systems not carrying these symbols must be processed with the <sup>ST</sup> mixing program.

**!** **Please note that casting materials which are processed with the wrong mixing program will not meet the quality standards guaranteed by the respective manufacturer.**

The following points apply when selecting the mixing program:

Selecting the mixing program:

⇒ Activate the mixing program by pressing the corresponding program key (15, Fig. 7).

- ◆ The LED next to the program key goes on.

For mixing program the following applies:

- When the extrusion process has been completed, the mixing program selected remains in standby mode and is still enabled when the unit is switched off.
- When changing the cartridge, the pressure discs have been moved to their home position, the mixing program is deactivated.
- As long as the pressure discs are in their home position, the unit cover is open and no mixing program is activated, all program key LED are flashing.

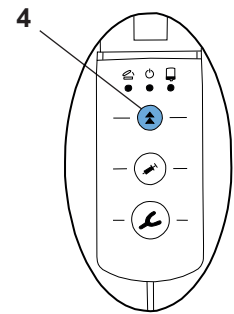
Fig. 5

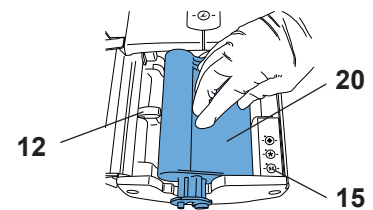
Fig. 6

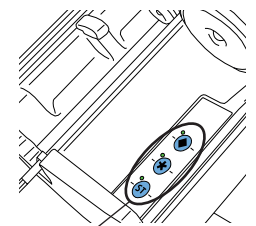
Fig. 7

- When the unit cover is closed, the LED will turn off. When opening the unit cover and no extrusion process has been started or no mixing program has been activated, they will start flashing again.
- If a button is pressed, such as extrude (impression material (2, Fig. 8) or dispense (3, Fig. 8), without having previously selected a programme, the device will automatically activate the  $\text{ST}$  programme. The LED next to the  $\text{ST}$  key goes on.
- The operator can change from one mixing program to a different one at any time by pressing the relevant program key.

## 5.2 Extrusion

- ⇒ Set up the unit as instructed in section 5.1.
  - ⇒ Press and hold down the advance key (2) or (3) (depending on the desired speed or function):
    - Impression tray (2), fast extrusion speed  
or
    - Dispenser (3), slow extrusion speed
  - ◆ The mixing hexagon (13, Fig. 1) starts to turn and threads its way automatically into the mixing nozzle (21, Fig. 1) and then stops again.
  - ◆ The pressure discs advance at high speed until they reach the impression material.
  - ◆ Once it comes into contact with the impression material the device will automatically change to the selected pressing speed and the mixing shaft begins to turn.
- ⇒ Fill the impression tray or dispenser.
- ⇒ Release the advance key as soon as the desired amount of material has been extruded.
- ◆ The pressure discs retract with a short relief stroke to prevent any additional impression material from being extruded.

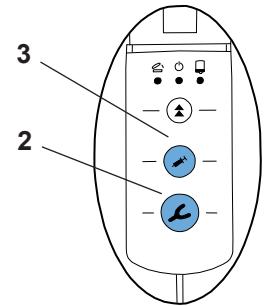
Fig. 8

**i** *The pressure discs will only move if the unit cover is closed.*

**i** *The threading of the mixer shaft into the mixing nozzle produces an audible noise. This is perfectly normal.*

**i** *The first 5 cm of material from a newly installed cartridge or from an again inserted used cartridge should be discarded. Please note the material manufacturer's recommendations!*

**i** *If the cartridge is emptied during an extrusion, the pressure discs automatically return to their home position so that a new cartridge can be quickly installed to start a new the extrusion process.*

## 5.3 Fill Level Indicator and Remaining Amount

### 5.3.1 Fill Level Indicator

The fill level indicator (1) helps you to estimate the amount of impression material remaining in the cartridge.

However, after installing a new cartridge or if the cartridge is very full, the red indicator will at first not be visible.

The indicator will only become visible once the pressure discs have advanced approx. 25 mm into the cartridge.

### 5.3.2 Remaining Amount Detection

The Dynamix<sup>®</sup>speed 2 is equipped with an electronic fill level detection.

Once there is only sufficient material remaining in a cartridge to fill approx. one casting spoon, the yellow "Residual level" LED (5) goes on.

The indicator passes a residual level marking.

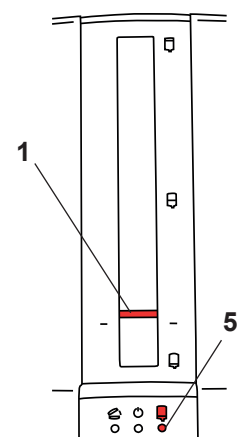
Fig. 9

## 6 Cleaning / Maintenance

The device does not contain any parts that require maintenance. Opening the device other than described in the instructions is not permitted!

Never use cleansers containing solvents.

Never use superheated steam for cleaning!

To clean the unit, merely wipe it down with a moist cloth.

### 6.1 Cleaning the Interior

- Cartridge bed (14, Fig. 1):

With the pressure discs (9, Fig. 1) retracted, the cartridge bed is easily accessible for cleaning.

- Pressure discs (9, Fig. 1) and spindle covers (10, Fig. 1):

In order to remove deposits and residues from the pressure plates or the spindle cover, it is possible to move the pressure plates into a suitable position whilst there is no cartridge in place and the device lid is closed.

**NEVER clean the spindles (11, Fig. 1) themselves. The grease applied to spindles is necessary for the unit's proper function.**

**Tip:**

**Remnants of impression material are most easily removed with a dry cloth or paper towel.**

In addition to wiping with a moist cloth, the following cleaning agents are approved:

- Incidur Spray (Ecolab)
- Incides N wipes (Ecolab)

Ensure adequate ventilation after using these agents to prevent an explosion-prone area.

### 6.2 Pressure Disc Replacement

The pressure discs (9, Fig. 1) are subject to wear as a result of their rubbing against the inner cartridge wall (20, Fig. 1). Highly abraded pressure plates can damage the foil bag within the cartridge (20, Fig. 1).

Furthermore, a wrongly placed cartridge can cause the pressure plate to break. Replacement pressure discs are therefore available as spare part. The spare parts are supplied with the necessary tool and installation instructions.

### 6.3 Unit Cover Replacement

The unit cover (8, Fig. 1) is available as spare part.

The spare parts are supplied with the necessary tool and installation instructions.

### 6.4 Fuse Replacement

**Before replacing the fuse, disconnect the unit from the mains power supply by unplugging the power cord from the wall outlet.**

Only replace fuses with new ones that have the same rating as those being replaced. Refer to the „Technical Specifications“ section.


**Never use fuses with a higher rating.**

⇒ Unplug the unit.

⇒ Pull the fuse holder (17) out.

⇒ Remove the defective fuse and insert a new one.

⇒ Push the fuse holder back in, making sure it is completely seated.

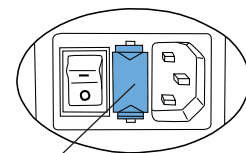17

Fig. 10

### 6.5 Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at [www.renfert.com/p918](http://www.renfert.com/p918).

Enter the following item number: 25 6200 0000.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list. Serial number and date of manufacturing are shown on the type plate of the unit.

# 7 Troubleshooting

en

Error	Cause	Corrective action
LEDs (5, 6, 7, Fig. 1) do not light, pressure discs will not move.	<ul style="list-style-type: none"> <li>The unit is not switched on.</li> </ul>	<ul style="list-style-type: none"> <li>Switch the unit on at the main power switch (16, Fig. 1).</li> </ul>
The unit does not switch on.	<ul style="list-style-type: none"> <li>Power cord not properly connected to the unit.</li> <li>Fuse blown.</li> <li>No power at the wall outlet.</li> <li>Unit fault.</li> </ul>	<ul style="list-style-type: none"> <li>Check that the unit plug is properly seated (Fig. 2).</li> <li>Replace the fuse (refer to Sec. 6.4).</li> <li>Check that there is power at the wall outlet.</li> <li>Return the unit for repair.</li> </ul>
Pressure discs will not move, "Unit cover" LED (7, Fig. 1) is on or flashes.	<ul style="list-style-type: none"> <li>Unit cover not properly closed.</li> </ul>	<ul style="list-style-type: none"> <li>Close the unit cover (8, Fig. 1) and check that the cartridges are properly seated. If required, seat cartridges correctly.</li> </ul>
Pressure plates move very slowly towards the cartridge.	<ul style="list-style-type: none"> <li>The cover was opened before the pressure plates stopped. A previous cartridge position has been saved.</li> </ul>	<ul style="list-style-type: none"> <li>Press retract key (4, Fig. 1) and wait until the pressure plates have reached their home position and have come to a complete stop. Then repeat the process with the cartridge.</li> </ul>
Cartridges cannot be removed / cannot be installed.	<ul style="list-style-type: none"> <li>Pressure discs have not fully retracted to their home position.</li> </ul>	<ul style="list-style-type: none"> <li>Press the retract key (4, Fig. 1) and allow the pressure discs to retract until they stop automatically. They are now in their home position (refer to Sec. 5.1).</li> </ul>
A loud sound can be heard when the mixer shaft threads itself into the mixing nozzle.	<ul style="list-style-type: none"> <li>A spring pushes the mixer shaft forward until it reaches a stop.</li> </ul>	<ul style="list-style-type: none"> <li>This is perfectly normal and requires no intervention.</li> <li>Only install the mixing nozzle subsequently.</li> </ul>
Mixing nozzle (21, Fig. 1) is not turning.	<ul style="list-style-type: none"> <li>The mixer shaft has not threaded itself.</li> <li>Material has hardened in the mixing nozzle.</li> <li>The mixer motor's thermal protector has activated.</li> </ul>	<ul style="list-style-type: none"> <li>Check that the cartridge is properly seated.</li> <li>Install a new nozzle.</li> <li>Allow the unit to cool down for approx. 30 minutes, Observe the maximum permissible operating time (2 minutes of operation, 5 minute pause).</li> </ul>
Unit automatically reduces the Extrusion speed.	<ul style="list-style-type: none"> <li>Unit detects a highly viscous impression material and automatically reduces the feed rate and mixer rotary speed.</li> <li>Contamination (e.g. debris) in the cartridge holder.</li> <li>Insufficient spindle lubrication.</li> <li>The pause time was not observed.</li> </ul>	<ul style="list-style-type: none"> <li>This only occurs if the material is highly viscose or is cold. Refer to the manufacturer's processing instructions (processing temperature).</li> <li>Clean the inner surfaces of the cartridge holder and pressure plate.</li> <li>Lubricate the spindles (special grease!).</li> <li>At least 5min intermission.</li> </ul>
The mixing nozzle bursts (21, Fig. 1).	<ul style="list-style-type: none"> <li>Incorrect mixing nozzle installed.</li> <li>Mixing tip/ foil bag not compatible.</li> </ul>	<ul style="list-style-type: none"> <li>Use the mixing nozzle specified by the manufacturer.</li> <li>Contact the manufacturer of the mixing tip / foil bag.</li> </ul>
The tubular bag bursts or is damaged.	<ul style="list-style-type: none"> <li>Pressure discs are too worn or have been damaged, causing the tubular bag to be pinched between the pressure disc and the cartridge.</li> <li>The material has partially hardened in the cartridge.</li> </ul>	<ul style="list-style-type: none"> <li>Replace the pressure discs (refer to Sec. 6.2).</li> <li>Replace the cartridge (refer to Sec. 5.1).</li> </ul>
Advance stops for no apparent reason and the unit performs a short relief stroke.	<ul style="list-style-type: none"> <li>Material has hardened in the cartridge.</li> </ul>	<ul style="list-style-type: none"> <li>Use new mixing nozzle.</li> </ul>
Advance stops for no apparent reason and the pressure discs return to their home position.	<ul style="list-style-type: none"> <li>The controller has detected an overload.</li> <li>Material has hardened in the cartridge.</li> <li>Insufficient spindle lubrication.</li> </ul>	<ul style="list-style-type: none"> <li>Impression material is too cold, Refer to the manufacturer's processing instructions.</li> <li>Use a new cartridge.</li> <li>Lubricate the spindles (special grease!).</li> </ul>
The mixing program LEDs are flashing while the unit cover is open.	<ul style="list-style-type: none"> <li>No mixing program is selected.</li> </ul>	<ul style="list-style-type: none"> <li>Select the correct mixing program for the casting material (see section 5.1.1).</li> </ul>

## 8 Technical Specifications

Item number:	25 6200 0000	25 6200 1000	25 6200 3000
Mains voltage:	230 V	120 V	100 V
Mains frequency:	50/60 Hz	60 Hz	50/60 Hz
Power consumption:	280 VA		
Mains input fuse:	2 x T2AH	2 x T3,15 AH	
Dimensions (height x width x length):	330 x 190 x 380 mm (13 x 7,5 x 15 inch)		
Weight (empty):	7,0 kg		
Noise level:	< 70 dB(A)		
Protection class:	I		

## 9 Warranty

Provided the unit is properly used, Renfert warrants the all components of the Dynamix®*speed* 2 mixing unit for a **period of 3 years**.

Warranty claims may only be made upon presentation of the original sales receipt from the authorized dealer. Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee.

These parts are marked in the spare part list.

The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or; in case of unusual influences or influences not in compliance with the utilization instructions.

Warranty service shall not extend the original warranty.

## 10 Disposal Information

### 10.1 Disposing of Consumables

Empty cartridges and tubular bags as well as used mixing nozzles must be disposed of in accordance with the manufacturer's instructions.

### 10.2 Instrument Disposal

The instrument must be disposed of by a specialist facility. This specialist facility must be informed of any hazardous residue in the instrument.

#### 10.2.1 Disposal Information for EU Nations

In order to maintain and protect the environment, to prevent environmental contamination and to improve the reutilisation of raw materials (recycling), the European Commission has established guidelines according to which manufacturers must take back electrical and electronic devices in order to forward them to a regulated disposal or recycling system.

**Within the European Union, devices identified with this symbol may therefore not be disposed of with unsorted municipal solid waste.**

For more information regarding proper disposal please apply at your local authorities.

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# 2 Information About Electromagnetic Compatibility (EMC)

## 2.1 General Information

- ▶ The product / equipment was tested in accordance with DIN EN 60601-1-2.
- ▶ The product / equipment has no essential performance characteristics as per the definition in IEC 60601-1-2.
- ▶ The product / equipment satisfies Class A according to CISPR11:  
The properties of this equipment determined by emissions allow its use in industrial applications and in hospitals (CISPR11, Class A). If use in residential applications (for which typically Class B is required according to CISPR11), the equipment may not provide adequate protection from radio services. The user may need to take remedial measures such as repositioning or realigning the equipment.
- ▶ When using the product / equipment in a **SPECIAL ENVIRONMENT**, its function may not be guaranteed nor can it be ensured that it satisfies the **ESSENTIAL PERFORMANCE CHARACTERISTICS**.
- ▶ **SPECIAL ENVIRONMENTS** may include, for example:
  - Military areas (e.g. a submarine, close to radar equipment or weapons control systems)
  - Areas of heavy industry
  - Medical treatment areas with high power medical electrical (ME) systems (e.g., RF surgical equipment, magnetic resonance imaging equipment).
- ▶ Operating in an **PROFESSIONAL HEALTH CARE FACILITY ENVIRONMENT** is permitted provided an adequate distance is kept to the high power ME systems.
- ▶ The product / equipment must not be placed and operated directly next to other electrical equipment. If this is unavoidable, the function of the product / equipment and the other electrical equipment should be checked to rule out any malfunctioning caused by electromagnetic interference.
- ▶ The use of accessories other than those provided for in the operating instructions, however they may have an adverse effect on the EMC properties.
- ▶ Stacking products / equipment is not permitted.
- ▶ Portable and mobile communication devices (e.g. cell phones, radio equipment) can affect the functioning of the product / equipment.  
Maintain a distance of at least 30 cm (12 inches) between the product / equipment and mobile communication devices.

## 2.2 Interference Emission and Immunity Levels

### 2.2.1 Maintaining the Interference Emission Level

Interference Measurement	Compliance
Conducted interference (interference voltage) CISP11	Class A Limit values are complied with.
Radiated interference variable (noise field intensity) CISP11	Class A Limit values are complied with.
Distortion due to harmonics IEC 61000-3-2	Class A Limit values are complied with.
Voltage fluctuations and flicker IEC 61000-3-3	Satisfied

### 2.2.2 Maintaining the Interference Immunity Level

Interference Immunity Test	Standard / Test Level	Match Level
Static electricity discharge	IEC 61000-4-2 Air discharge 15 kV maximum Contact discharge: 8 kV maximum	15 kV 8 kV
High-frequency electromagnetic fields 80 MHz - 2.7 GHz	IEC 61000-4-3 10 V/m	10 V/m
High-frequency electromagnetic fields (near field) 380 MHz - 5.8 GHz	IEC 61000-4-3 28 V/m maximum	No interference
Electrical fast transients (burst)	IEC 61000-4-4 2 kV	2 kV
Surge voltages (surge)	IEC 61000-4-5 1 kV	1 kV
Conducted disturbances 150 kHz - 80 MHz / ISM + AR	IEC 61000-4-6 3 V / 6 V	3 V / 6 V
Magnetic fields with energy frequency 50 / 60 Hz	IEC 61000-4-8 30 A/m	30 A/m
Mains voltage dips and short interruptions	IEC 61000-4-11 Test voltage / exposure time 70 % / 0.5s 0 % / 0.02s 0 % / 0.01s 0 % / 5s	The response to the mains voltage dips and short interruptions is device specific. The behavior of the product / equipment in these scenarios is specified in the operating instructions for the product.

## 2.3 Unknown EMC Environment

Avoid operation in the vicinity of stationary transmitters such as base stations of radio telephones, land mobile radios, amateur radio stations, AM and FM broadcasting and television transmitters.

When operating in this kind of environment, pay attention to the behavior of the product / equipment.

Consider changing the location if there are any unusual functional limitations or reduced performance characteristics.

The field strength measured at the site should not exceed the above referenced compliance levels.

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

**[www.renfert.com](http://www.renfert.com)**

Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany  
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70  
[www.renfert.com](http://www.renfert.com) • [info@renfert.com](mailto:info@renfert.com)

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA  
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87  
[www.renfert.com](http://www.renfert.com) • [info@renfertusa.com](mailto:info@renfertusa.com)  
USA: Free call 800 336 7422