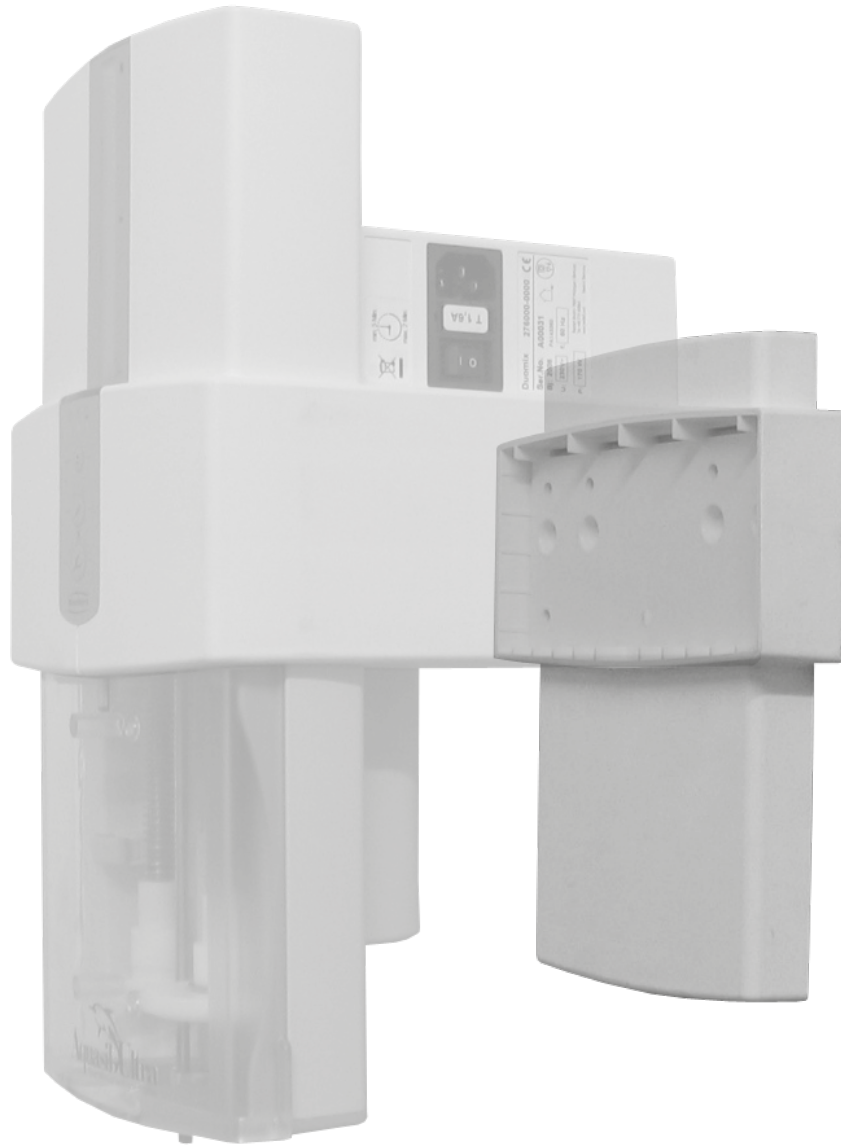
making work easy

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Symbols

Danger

This symbol indicates a impending injury risk. Please refer to the operating instructions.

Attention

Failure to observe this information can result in damage to the unit.

Standard delivery

Refer to Figure 1:

(A) 1x base for wall mounting

(D) 3x dowels

(B) 1x wall mounting plate

(E) 2x self-tapping screws

(C) 3x fastening screws for the wall mounting plate, with washers

(F) 1x Allen key, SW3

Mounting

The wall on which the support is to be mounted must be sufficiently strong!

Please ensure that there are no electric cables or water pipes running through the wall at the place of installation!

Please have the following at hand:

- A pencil;
- A Philips head screwdriver;
- A power borer;
- 8 mm [0.3152 inch] bore bit, appropriate for the wall material to be drilled;
- The wall support, incl. mounting and fastening hardware.

Proceed as follows to install the wall support:

⇒ Unplug the unit from the mains outlet.

Install the wall mount base on the unit:

⇒ Lay the unit on its side.

⇒ Remove the 6 screws securing the unit's base (Figure 2).

⇒ Remove the unit's base (Figure 3).

⇒ Install the wall support base and secure it with the 6 screws (Figure 4).

Be sure to use all 6 screws!

Install the wall mounting plate:

⇒ Using the wall mounting plate as a template, mark the required bore holes (Figure 5).

⇒ Bore the necessary mounting holes (8 mm [0.3152 inch] bit >>> at least 55 mm [2.167 inches] deep) + insert the dowels into the holes.

⇒ Secure the wall mount with fastening screws including washers (C) (Fig. 6).

Hang the unit in the holder:

◆ To hang the unit, the 4 hooks on the wall mounting plate slide into the 4 slots on the bottom of the unit base (Figure 7).

◆ The power cord can be fed through the openings on the bottom of the base (Figure 7).

Take care not to pinch the power cord when hanging and mounting the unit!

⇒ Hang the unit on the wall mounting plate by mating the slots on the bottom of the unit with the hooks on the wall mounting plate (Figure 8).

⇒ Swivel the unit towards the wall (Figure 9).

⇒ Install the self-tapping screws (E) to secure the unit to the wall mounting plate (Figure 10).

⇒ Reconnect the unit to the mains power supply.

REACH

Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany