


CARA PRINT CUBE

QUICK START GUIDE

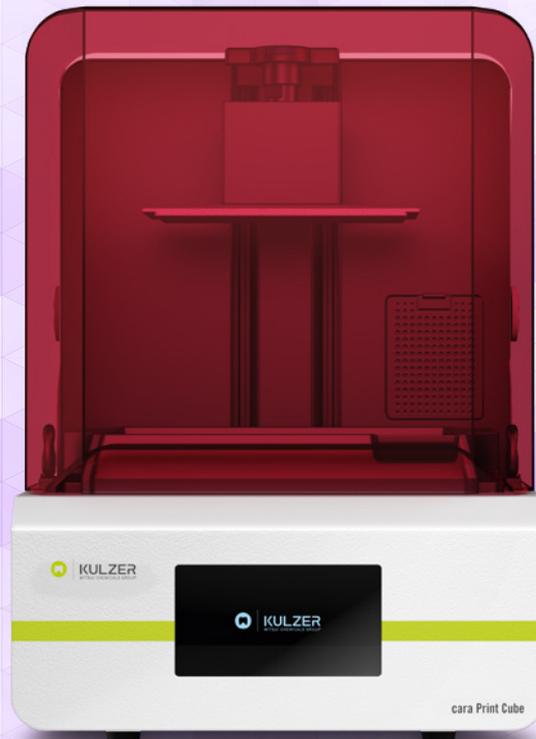

[kulzer.com/
mycube-support](https://kulzer.com/mycube-support)



KULZER
MITSUI CHEMICALS GROUP

INFORMATION

01 Equipment

02 Installation


03 Preparation before printing

04 Printing


05 Technical Support

01 EQUIPMENT


Accessories

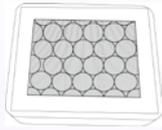

Air filter

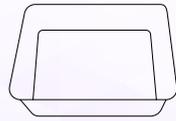


Plastic box-big

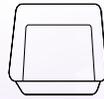


Plastic box-small




Spray bottle

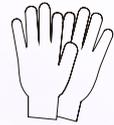

Disposable gloves

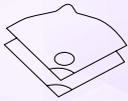

Disposable filter

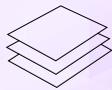


Dust-free wipes



Plastic nippers




Shovel blade

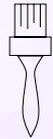


Brush



Allen wrench

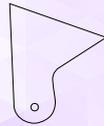


Scraper

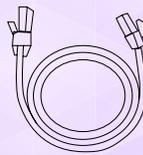

Ethernet cable



Power cable

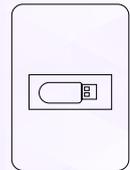

USB drive



Carpenter knife



Silicone scraper



Ceramic hob scraper



Hammer




Tray




Screen protector




Release film

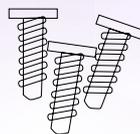

Spare screws

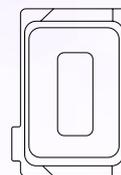


Resin tank cover

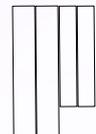

Tape

NOTE:
 PICTURES ABOVE ARE A DEMO FOR TOOLS AND ACCESSORIES PACKED WITH THE PRINTER.
 THE ACTUAL TYPE OR QUANTITY OF THESE ITEMS MAY BE ADJUSTED WITHOUT NOTIFICATION.

02 INSTALLATION

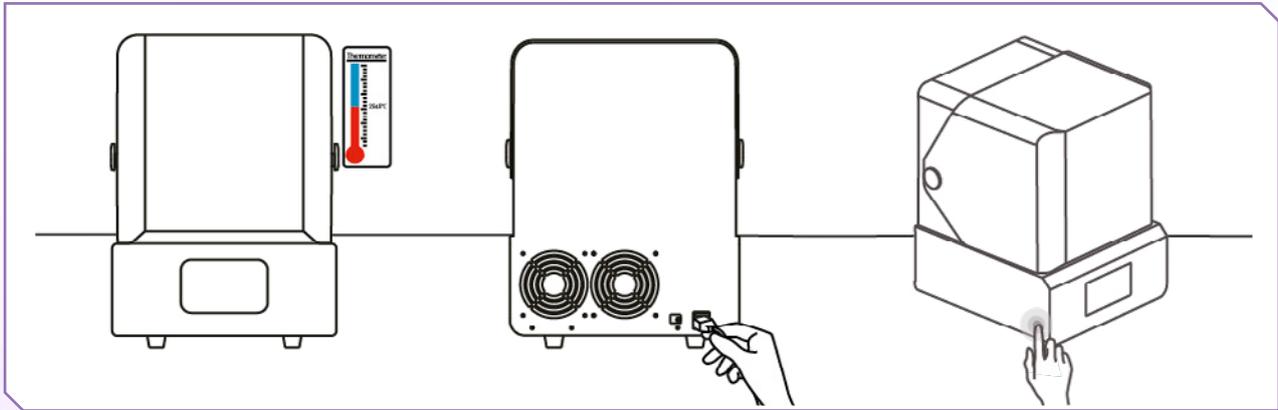


1. Place the printer on a stable platform. Avoid direct sunlight and dust pollution, and maintain the ambient temperature at 18 °C to 30 °C

2. Plug in the power cord (Type F for Germany, France, Italy and Korea, Type B for USA and Canada, Type I for Australia and New Zealand, Type N for Brazil)


3. Turn on the printer

NOTE:

PLEASE DO NOT DISPOSE THE ORIGINAL PRINTER PACKAGING, AS THIS MUST BE USED IN CASE A PRINTER BACK TRANSPORT IS NEEDED IN COMBINATION WITH A WARRANTY CLAIM.

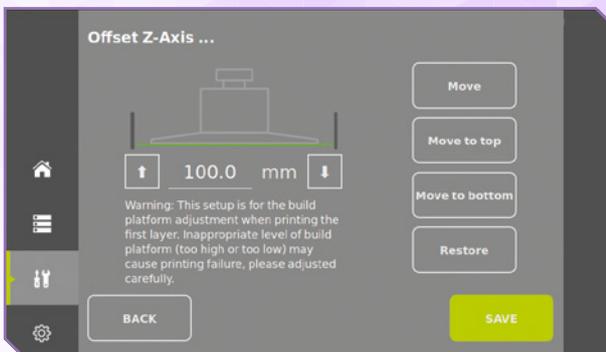


4. Click “Settings” – “Z-Axis Offset” – “Move to Top”

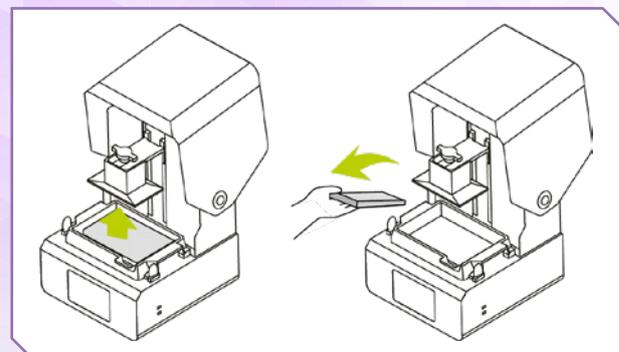

5. Once the platform is raised, remove the foam

6. In order to check if the build platform is still levelled after transport, remove the resin tank and place a piece of A4 paper (80 g/m³) on the LCD panel. Click “Move to bottom”. The build platform will move downwards until it barely contacts the surface of the LCD panel (maximum Z-axis travel).

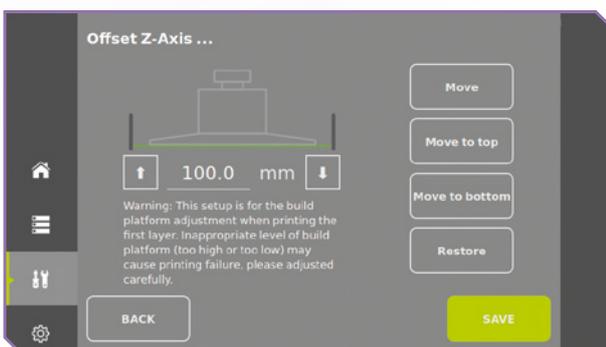

 When the building platform moves downwards, do not place your hands or any other parts of the body under the platform to avoid injury.

Drag the A4 paper (80 g/m³) outwards slowly to check if there is frictional feeling during the pulling process.


 Every time the building platform is changed, the platform intended to be used for the next print must be levelled before the first use.

 Please find more detailed information about using and maintaining the cara Print Cube printer here:



kulzer.com/
mycube-support

NOTE:
PLEASE REFER TO THE VIDEO “CARA PRINT CUBE – LEVELLING THE Z-AXIS” ON OUR SUPPORT HUB.

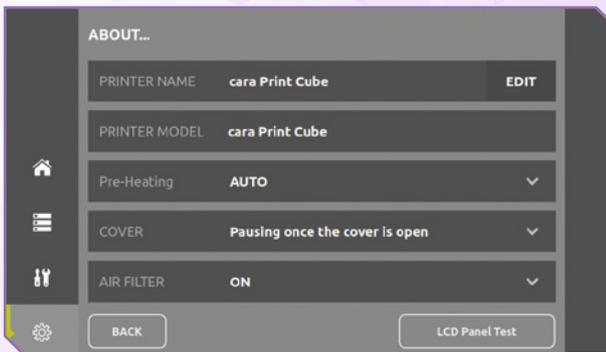


7. Click “Settings” – “Devices” – “LCD Panel Test”



8. Check if the projected light point matrix is partially missing, blinking or shaking. The light points should be projected as sharp points. Place a piece of A4 paper on the screen for better observation. Please wear appropriate eye protection to protect yourself against UV light


NOTE:
PLEASE PAY ATTENTION TO THE FILES ON THE DELIVERED USB HARD DRIVE. AMONGST OTHERS, THIS USB DRIVE CONTAINS THE USER MANUAL IN VARIOUS LANGUAGES.

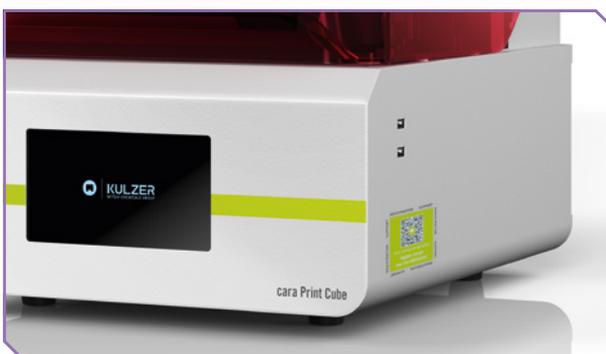

9. Register your printer using the following link: kulzer.com/mycube-registration or by scanning the QR code on the side of the printer. You will receive your free unlimited cara Print Cockpit/CAM license after successful registration.

03 PREPARATION BEFORE PRINTING

1

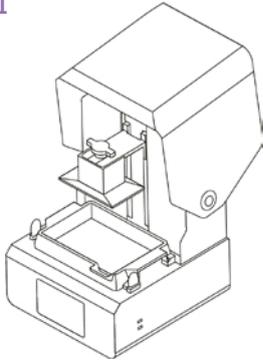


Check the printer if there is anything abnormal before printing

2




Wear rubber gloves for protection purposes

3

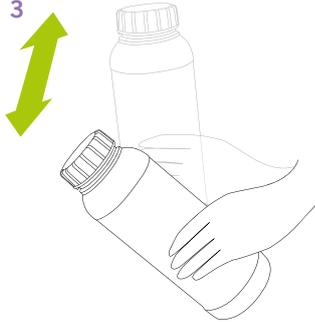


Shake the resin bottle according to the material instruction for use before printing to mix the resin in the bottle evenly

4

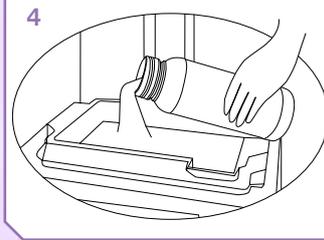


Pour a sufficient amount of resin into the resin tank. Before, please make sure that the resin tank is clean and not damaged. The photopolymer in the tank should not exceed the maximum line

5

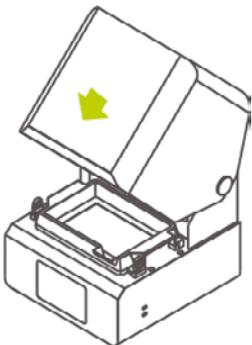


Close the cover

6

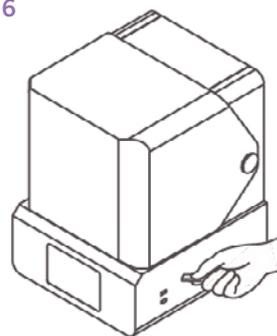


Insert a USB hard drive with a .capc file. The file must be created in the cara Print Cockpit software before


04 PRINTING

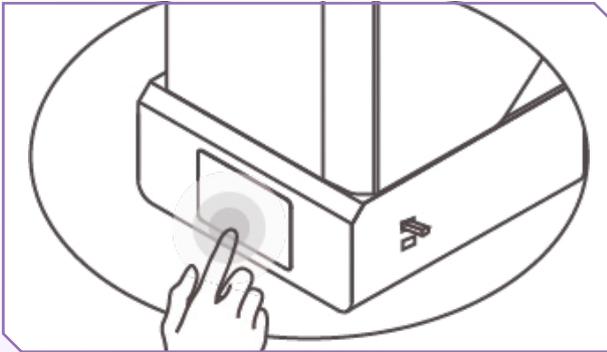

1. Click on the touch screen to proceed the printing task.

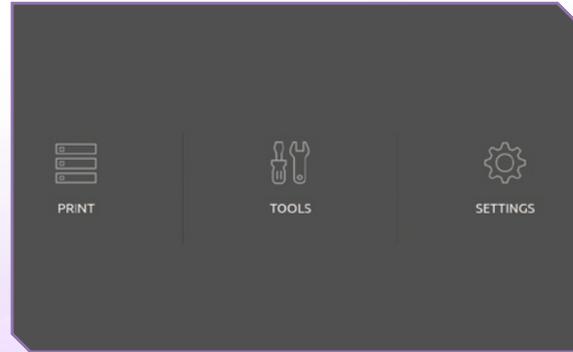

2. Click "Print"

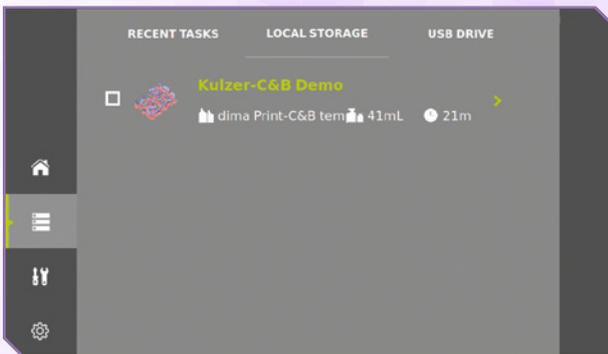

3. Click "USB Drive" first, then select the targeted print job and click ">" to load it



4. After confirming that the print information is correct, click "Print"

05 TECHNICAL SUPPORT

If you need support during the use of cara Print products, please contact your local Kulzer representative.

Please see a list of contacts below.

BRASIL

Kulzer South America Ltda.
Rua Cenno Sbrighi,
27 – Sala 42 – Água Branca
05036-010 São Paulo SP
Brasil
support.br@kulzer-dental.com

ITALY

Kulzer S.r.l.
Via Console Flaminio 5/7
20134 Milano
Italy
support.it@kulzer-dental.com

USA & CANADA

Kulzer, LLC
4315 S. Lafayette Blvd.
South Bend, IN 46614
USA
cara-service-na@kulzer-dental.com

GERMANY

Kulzer GmbH
Leipziger Straße 2
63450 Hanau
Germany
cara@kulzer-dental.com

REPUBLIC OF KOREA

Kulzer Korea Co., Ltd.
Room 501,
Namsung Plaza Bldg.
345-30 Gasan-dong,
Geumcheon-gu, Seoul 153-782
Republic of Korea
support.kr@kulzer-dental.com

AUSTRALIA & NEW ZEALAND

Kulzer Australia Pty Ltd
New Unit 20/53 Lorraine St,
PEAKHURST, NSW, 2210
Australia
support.anz@kulzer-dental.com

FRANCE

Kulzer France S.A.
Les Conquérants – Bât. Everest
1, avenue de l'Atlantique
91976 Les Ulis –
ZA Courtabœuf Cedex
France
support.fr@kulzer-dental.com

**FOR
MORE INFORMATION
VISIT KULZER.COM**



[kulzer.com/
mycube-support](http://kulzer.com/mycube-support)



KULZER
MITSUI CHEMICALS GROUP

Contact in Germany

Kulzer GmbH
Leipziger Straße 2
63450 Hanau, Germany

05 TECHNICAL SUPPORT

If you need support during the use of cara Print products, please contact your local Kulzer representative.

CONTACT:

Kulzer GmbH
Leipziger Straße 2
63450 Hanau
Germany

Service Tel.:
or mail to ...@kulzer-dental.com



**FOR
MORE INFORMATION**

VISIT **KULZER.COM/...**
FOR A COMPLETE OVERVIEW.



[kulzer.com/
mycube-support](https://kulzer.com/mycube-support)



KULZER
MITSUI CHEMICALS GROUP

Contact in Germany

Kulzer GmbH
Leipziger Straße 2
63450 Hanau, Germany